Hypromellose Eye Drops
0.3% w/v 10ml

PRESERVATIVE FREE IN THE EYE

What is Xailin Hydrate Eye Drops?

Xailin Hydrate Eye Drops is a multidose eye drop for the immediate relief of dry eye sensations. The pack contains 10ml of sterile eye drops.

Composition:

Xailin Hydrate Eye Drops contains Hypromellose 0.3% w/v, sodium perborate & aqueous buffered vehicle q.s.

Xailin Hydrate contains a disappearing preservative that turns into water and oxygen upon contact with your eye, thereby minimizing the irritation that may be caused by traditional preservatives while still providing the convenience of a multidose form. All you feel is the soothing relief of the eye drop.

When should it be used?

Xailin Hydrate Eye Drops is used to lubricate and protect the eye and provide immediate relief of dry eye sensations. Xailin Hydrate Eye Drops provides soothing relief from the symptoms of dry eye sensations, such as soreness, irritation or a gritty sensation. Dry eye symptoms occur as a result of ocular surface disorders that can be caused by a number of different reasons where the normal tears are either of poor quality or produced in too little quantity to keep the eye sufficiently moist. Xailin Hydrate can be used by contact lens wearers.

How to use:

The usual dosage is one or two drops to be applied two to four times a day as needed, or as directed by your healthcare practitioner who may advise you to use this product in a different way. No dosage adjustment is required in children.

Before opening a new bottle of Xailin Hydrate Eye Drops, please check that the safety ring on the cap of the bottle is unbroken.

1. Rotate the cap anticlockwise to break the seal.
2. Wash and dry your hands well before use.
3. Remove the cap.
4. Tilt your head back and look up.
5. Gently pull down the lower lid of the eye to form a small pocket between your eyelid and your eye.
6. Hold the bottle between your thumb and middle finger of your other hand and with your index finger apply moderate pressure to base of the eye drop bottle to release one drop of Xailin Hydrate Eye Drops into the pocket.
7. Blink your eye a few times.
8. If you think that you have missed the eye, then insert another drop.

EN Instruction for use

It is important to read this leaflet carefully before using Xailin Hydrate Eye Drops. The information provided summarises the important points about the product but if you have any further questions then please ask your doctor, pharmacist or optician.

9. Carefully wipe away any excess Xailin Hydrate Eye Drops around the eyelid with a tissue
10. Repeat the process for the other eye, if required.
11. Replace the cap on the container, trying not to touch the applicator tip with anything, including the eye or your fingers.
12. Wash your hands well after use.
13. Leave at least 30 minutes between administrations of any other ophthalmic drugs.

If you are unsure about how to use Xailin Hydrate Eye Drops or if your eye condition persists, consult your doctor, pharmacist or optician.

Precautions and warnings:

- Do not use if the safety ring on the cap of the bottle is broken.
- Do not use if you are allergic to the product.
- Do not touch the tip of the bottle with your eye or any surface.
- Do not share Xailin Hydrate Eye Drops with any other person.
- Xailin Hydrate Eye Drops can cause blurring of vision after application. Wait until your vision is clear before driving or using machinery.
- Do not use after the expiry date shown on the carton or printed on the bottle.
- Store between 1°C and 25°C.
- Discard the contents 28 days after first opening.
- Keep out of the reach and sight of children.
- External ocular use only, do not swallow.
- Consult your doctor, pharmacist or optician before you use Xailin Hydrate Eye Drops if you are pregnant or breast-feeding.
- Dispose of the used bottle according to your local waste management regulations.

nicox Distributed by Nicox Pharma, CS10313, Sophia Antipolis, 06560 Valbonne, France - nicoxpharma@nicox.com
Medicom Healthcare Ltd., 235 Hunts Pond Road, Fareham, Hampshire, PO14 4PJ, U.K.

STERILE A Sterilised using aseptic processing techniques

Xailin is a trademark of Nicox
Last revision of the text: June 2014

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